




Voyager Web Viewer Installation on MAC




- Website: www.voyagerpacs.com
- Email: info@voyagerpacs.com
- Phone: +61 3 9811 9901
- Fax: +61 3 9811 9919

- 1.1. Open Safari and in the Address Bar type in <https://images.aspireradiology.com.au> hit Enter.
- 1.2. Click on the link 'For MAC Click Here', under Voyager DICOM Viewer (For Diagnostic Use).
- 1.3. A new page will load up and click on Link and the installer will start getting downloaded from the Server.

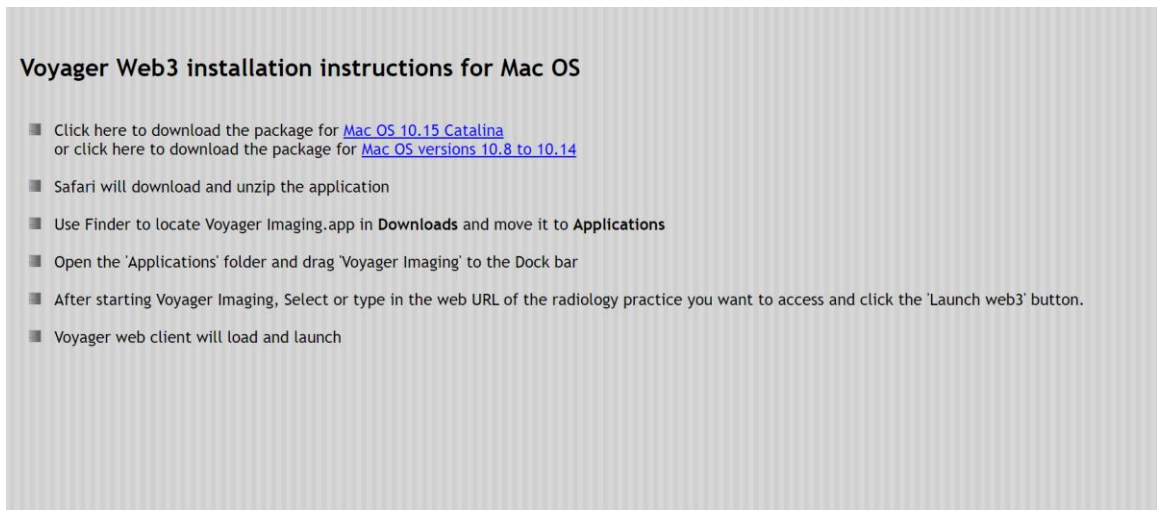

- 1.4. Once the installer is downloaded, double click 'VoyagerImaging.zip' in 'Downloads'. This will add Voyager Imaging under Devices.
- 1.5. Run Voyager Imaging and confirm to 'more to application folder'.
- 1.6. Open the 'Applications' folder and drag 'Voyager Imaging' to the Dock Bar.
- 1.7. Go to 'Dock Bar' and open 'Voyager Imaging'.
- 1.8. Voyager Web Launcher application will open up. Put the Server Name as <https://images.aspireradiology.com.au> and hit 'Launch'.

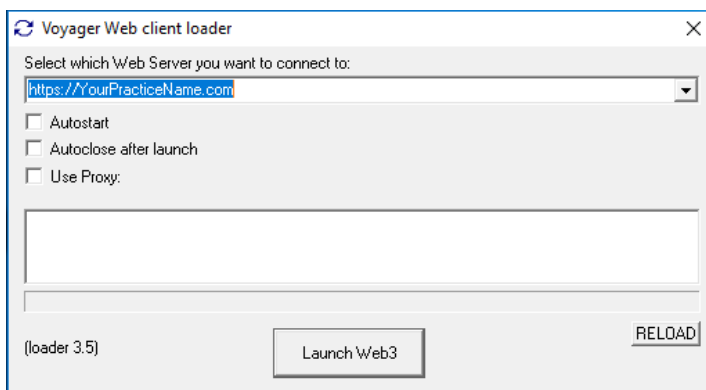

- 1.9. Once Voyager Web loads up, put in your Username and Password and hit 'Login'..
- 1.10. Voyager Web will log into the Server. Click on the relevant checkboxes on left or go to 'Quick Search' on the top to find your patient's images.

